


# Inline Filter Installation and Cleaning


## How to Install the Tube

### Step 1

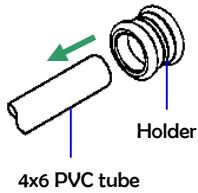


Put the tube holder around the tube

### Step 2

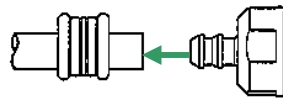


Insert the filter nozzle into the tube

### Step 3

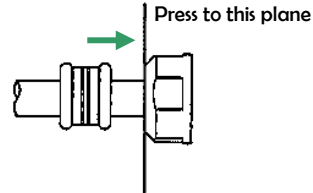

Press the holder towards the nozzle


### Step 4

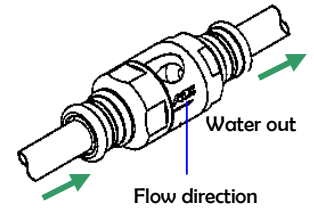

Proper installation should look like this


## How to Assemble the Filter Body

### Step 1

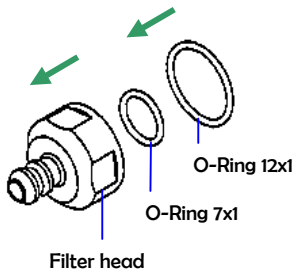


Install the O-rings

### Step 2

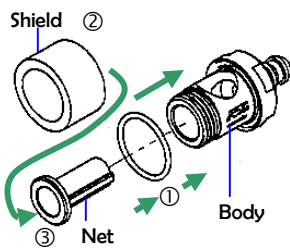


Assemble the O-ring, net, and shield in order: ①-②-③

### Step 3

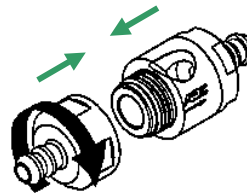

Tighten the filter head on the filter body


### Step 4

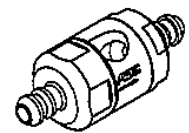

Make sure the filter head is in place and secure


## How to Clean the Filter Net

### Step 1

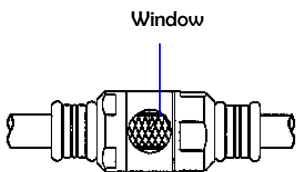


Inspect the net from the window for dirt

### Step 2

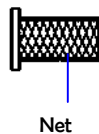

Disassemble the net from the filter


### Step 3

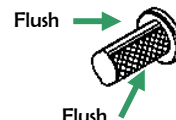

Clean the net

### Step 4

Make sure to place the net eye facing the window

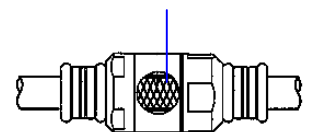

Re-install the net to the filter