

POLISHING HAND PIECE ASSEMBLY

For newer models of the straight nose cone



FIGURE A: Micro-motor



FIGURE B: Straight hand piece



FIGURE C: Propy head

The micro-motor, straight hand piece, and propy head are all components of the polishing hand piece.

To assemble:

1. Examine the colored strip (Figure B, Lock indicator) of the straight hand piece. Blue indicates that the straight nose cone is in the locked position. Red indicates that the hand piece is unlocked.
2. If locked, unlock the straight nose cone by turning the barrel (Figure B, Barrel) so that the lock indicator is red. The straight nose cone may be packaged with a place holder rod. After unlocking, remove the place holder from the straight nose cone.
3. Insert the stem of the propy head in the opening at the tip of the straight nose cone (Figure B, a).
4. Lock the straight hand piece to secure the propy head by turning the barrel (Figure B, Barrel) so that the lock indicator is blue.
5. Slide the base of the straight nose cone (Figure 4, b) over the micro-motor stem (Figure A, 1).